BRAUN

BP1750

BP1700

Type 6076

Type 6075

VitalScan/VitalScan Plus

English4

Español..... 14

Internet: www.kaz.com/braun

This product is manufactured by Kaz Inc under a license to the 'Braun' trademark. 'Braun' is a registered trademark of Braun GmbH, Kronberg, Germany.
©2009 Kaz Incorporated.

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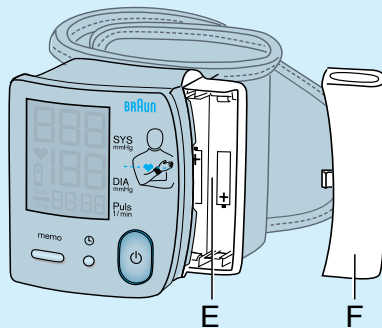
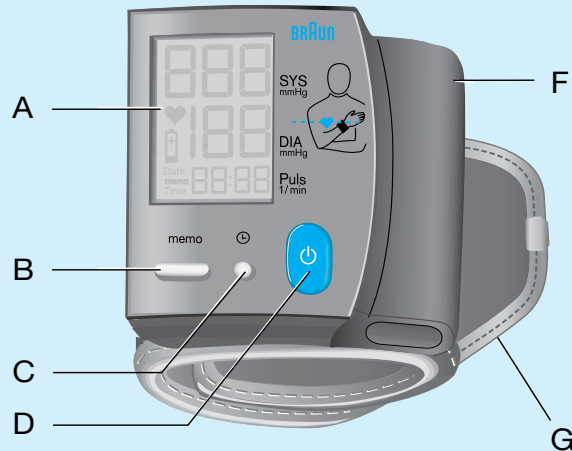
Product description

- A Display
- B Memory button («memo»)
- C Time/date button
- D Start/Stop button 
- E Battery compartment
- F Battery compartment cover
- G Cuff

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
English

Braun wrist blood pressure monitor has been developed for easy and accurate blood pressure measurements from the wrist. The measuring accuracy of Braun's wrist blood pressure monitor was tested at the time of manufacture and was proven by clinical research in accordance with AAMI/ANSI-SP10.

What you should know about blood pressure

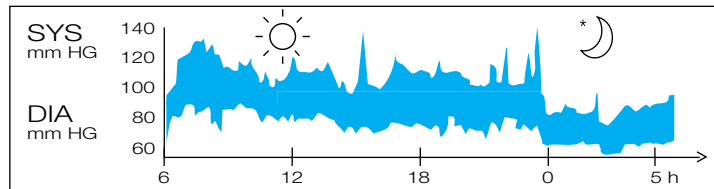Blood pressure constantly changes throughout the day. It rises sharply in the early morning and declines during the late morning. Blood pressure rises again in the afternoon and finally drops to a low level at night. Also, it may vary in a short period of time. Therefore, readings from successive measurements can fluctuate.

Blood pressure readings from a healthy 31-year-old male, measured at 5-minute intervals

Blood pressure measured in a doctor's office only provides a momentary value. Repeated measurements at home better reflect one's actual blood pressure values under everyday conditions.

Moreover, many people have a different blood pressure when they measure at home, because they tend to be more relaxed than when in the doctor's office. Regular blood pressure measurements taken at home can provide your doctor with valuable information on your normal blood pressure values under actual «everyday» conditions.

The World Health Organisation (WHO) has set up the following standard blood pressure values when measured at resting pulse:

Blood Pressure (mmHg)	Normal Values	Mild Hypertension	Severe Hypertension
SYS = systole (upper value)	up to 140	140-180	over 180
DIA = diastole (lower value)	up to 90	90-110	over 110

- To ensure accurate measuring results, carefully read the complete use instructions.
- This product is intended for household use only. Keep product and batteries away from children.
- People suffering from cardiac arrhythmia, vascular constriction, arteriosclerosis in extremities, diabetes or users of cardiac pacemakers should consult their doctor before measuring their blood pressure themselves, since deviations in blood pressure values may occur in such cases.
- If you are under medical treatment or taking any medication, please consult your doctor first.
- The use of this blood pressure monitor is not intended as a substitute for consultation with your doctor.

Inserting batteries

- Use alkaline batteries, type LR03 (AAA) as supplied with this product.
- Remove the battery compartment cover (F) and insert two batteries with correct polarity (see symbol in the battery compartment (E)).
- New batteries will provide approximately 200 measurements.

- Only discard empty batteries. They should not be disposed of in the household waste, but at appropriate collection sites or at your retailer.

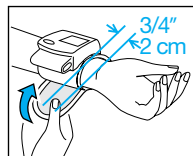
Key rules for accurate blood pressure measurement

- Always take readings at the same time of day, ideally in the morning, under the same conditions.
- Do not measure within 30 minutes after exercising, bathing, eating, smoking or consuming coffee, tea or alcohol.
- Take off wrist watch and jewelry before fitting the monitor on the measuring arm.
- Position the monitor 3/4" (2 cm) below the palm line.
- While taking a measurement, sit down, relax, keep still, especially the hand of the measuring arm and do not move or speak.
- Hold the monitor at heart level.
- Wait approximately 3 minutes before repeating a measurement.
- Do not use a cellular phone near the unit.


Taking a measurement


1. Fasten the monitor on the palm side of your wrist, roughly 3/4" (2 cm) below the palm line. Make sure that the cuff (G) fits tightly.

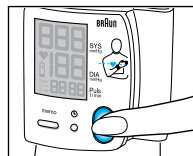2. Press the red Start/Stop button (D).


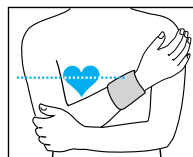
3. Hold the monitor at heart level. The measurement starts automatically. While measurement is in progress, do not move or speak.

4. At the end of the measurement, a beep sounds and the reading is displayed:

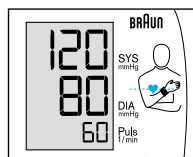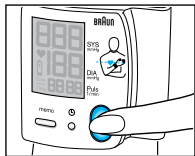Upper (systolic) value
Lower (diastolic) value
Pulse


5. Press the Start/Stop button  to turn the product off. Otherwise the product will turn off automatically after 2 minutes. If at any time you want to stop the measurement, press the red Start/Stop button .

Memory function

Your blood pressure monitor can store the readings of 90 measurements (BP1700 model: 60 measurements). Storing is done automatically after each completed measurement. The memory is nonvolatile. This means that you will not lose stored data when changing batteries. The memory cannot be cleared. Once the memory is full, the oldest values will be overwritten.

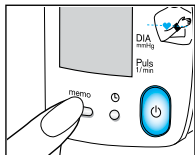The BP1750 model also features an average function to display the average reading of the last three measurements.


Recalling stored and average readings

- To recall stored and average readings, first turn the device off.
- Then press the memory button («memo») (B).

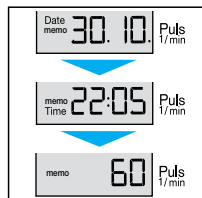BP1750: The average of the last three readings will be displayed. The lower display line shows «A-3», indicating that the average value is displayed.




BP1700: The last reading will be displayed. The lower display line successively shows date, time and pulse of the respective measurement.

- Repeatedly pressing the memory button will call up one by one all stored readings.
- Continuously pressing the memory button will scroll through all stored readings.
- To exit the memory mode, turn the device off by pressing the Start/Stop button .

Setting time and date

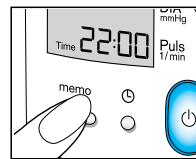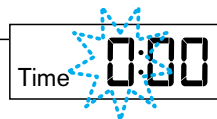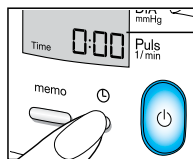- Push the time/date button (C). The lower display line shows «Time» and a flashing digit in the «hour» position. To adjust the hours, repeatedly push the «memo» button.

- Push button (C) again: The minutes digits will flash. Again, repeatedly push the «memo» button to adjust the minutes.

- Push button (C) again to adjust the date: The lower display line shows «Date» and flashing digits in the month position. Adjust the month by repeatedly pushing the «memo» button.
- Push button (C) again: The day digits will flash. Again, repeatedly push the «memo» button to adjust the day.
- To exit the time and date setting mode, push button (C).
- Otherwise the device automatically turns off after 2 minutes and activates the settings made so far.

Storage and cleaning

Clean the product with a soft, slightly damp cloth. Stains on the cuff (G) can be carefully removed with a damp cloth. Do not wash or dry clean the cuff. Do not store the product in a place exposed to direct sunlight or high humidity.

Calibration

This product has been calibrated at the time of manufacture. If used according to the use instructions, periodic recalibration is not required.

What to do if ...

Problem	Reason	Solution
	Battery power is low.	Insert new batteries.
ERR	Measurement / handling error	Repeat measurement as described under «Taking a measurement». If problem persists, contact 1-800-327-7226.

Problem	Reason	Solution
---	Inflation above 320 mmHg	Repeat measurement.
While measuring (during deflation of the cuff), you can hear a clicking sound with every detected heartbeat.	The device works with a special deflation procedure (stepwise deflation).	none
Readings from repeated measurements differ considerably.	Blood pressure is a fluctuating value. For healthy adults, deviations of 10 to 20 mmHg are possible.	none
	Cuff is not properly fitted.	Ensure that the monitor is fitted as follows: – on the palm side of your wrist – approx. 3/4" (2 cm) away from the palm line
	Readings were not consistently taken at heart level.	For each measurement, hold the monitor at heart level.
	Talking, coughing, laughing, moving etc. when measuring will influence the reading.	While measuring, relax, keep still, do not move or speak.

Specifications

Model:	BP1750 / BP1700
Operation principle:	Oscillometric method
Display:	Liquid Crystal Display
Measuring range:	0~320 mmHg (cuff pressure) 40~199 / min (pulse rate)
Blood pressure measurement:	20 mmHg (minimum diastolic value) 300 mmHg (maximum systolic value)
Laboratory accuracy:	± 3 mmHg (cuff pressure) ± 5% of reading (pulse rate)
Clinical accuracy:	According to AAMI-SP10 with auscultatory reference: < 5 mmHg systematic offset < 8 mmHg standard deviation
Inflation:	Automatic
Deflation:	Step valve
Power supply:	2 batteries, type LR03 (AAA)
Operating temperature:	50°F (+ 10°C) to 104°F (+ 40°C)
Storage temperature:	-4°F (- 20°C) to 140°F (+ 60°C)
Humidity:	Up to 85% RH
Cuff:	Suitable for wrist circumference of 5" (13 cm) to 8" (21 cm)
Clinical validation:	According to AAMI-SP10 with auscultatory reference. (Please note that a validation against intra-arterial measurements can lead to different results for devices which are calibrated against auscultatory reference.)

If device is not used within specified temperature and humidity ranges, the technical accuracy can not be ensured.

IMPORTANT

Read the operating instructions.

Classification:

- Internally powered equipment
- Type BF equipment

Portable and mobile RF communications equipment can affect MEDICAL ELECTRICAL EQUIPMENT.

Please do not dispose of the product in the household waste at the end of its useful life. Disposal can take place at your local retailer or at appropriate collection points provided in your country.

Warranty

Kaz grants a 5 year warranty on the product commencing on the date of purchase. Within the warranty period we will eliminate, free of charge any defects in the appliance resulting from faults in materials or workmanship, either by repairing or replacing the complete appliance as we may choose.

This warranty extends to every country where this appliance is supplied by Kaz or its appointed distributor.

This warranty does not cover: damage due to improper use, normal wear or use as well as defects that have a negligible effect on the value or operation of the appliance. The warranty becomes void if repairs are undertaken by unauthorized persons and if original Braun parts are not used. To obtain service within the warranty period:

Contact Us

Questions or comments?

Call toll-free: 1-800-327-7226

consumerrelations@kaz.com

www.kaz.com/braun

Consumer Relations

250 Turnpike Road

Southborough, MA 01772 USA

© 2009, Kaz, Incorporated - Made in China

Distributed by Kaz, Incorporated, Hudson, NY 12534 USA

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