

Vicks Vapourizer

Model V-188UK

Important Notice of Usage for medicine cup

(This section replaces the "INHALANT STEAM ACTIVATED "available on the "Use and Care Manual" page 2)

INHALANT STEAM ACTIVATED

KAZ INHALANT or VICKS VAPOSTEAM™


can be filled in the medicine cup.

Those products are not delivered with the Vapourizer but are available at your local retailer or on some websites.

You can use steam inhalants or medicine oils brands available in drugstores and pharmacies.

For best results, fill the medicine cup on the cap (located in the front of the steam outlet vents) with the steam inhalant or medicine oil product.

DO NOT put this oil or steam inhalant in the water tank, it must be put in the medicine cup and not inside the tank. Any oil brand or medicine into the water tank will damage the unit and void the guarantee. (Fig. 3) These inhalants are designed for use on one vapourizer cycle (12-14 hours). After vapourizer shuts off, empty the residue in the medicine cup and clean with a soft tissue. Add fresh inhalant after refilling water container.

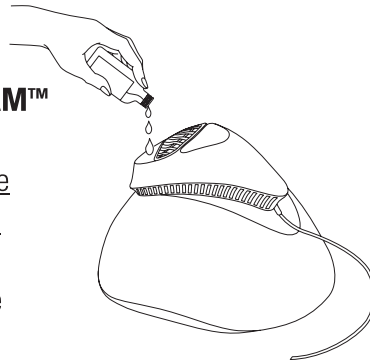

kaz

Kaz Europe SA
Place Chauderon 18
CH-1003 Lausanne - Switzerland
© 2010, Kaz Europe SA
www.kaz.com

Manufactured by Kaz Europe SA under a license to the Vicks trademark. Vicks is a registered trademark of The Procter & Gamble Company, Cincinnati, Ohio - USA. Vaposteam™ is a registered trademark of Procter and Gamble Inc.

Made in China

P/N: 311S/188/91