

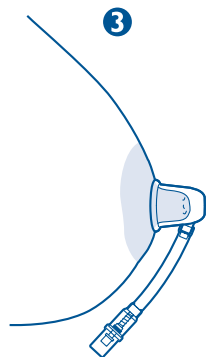
**AVENT**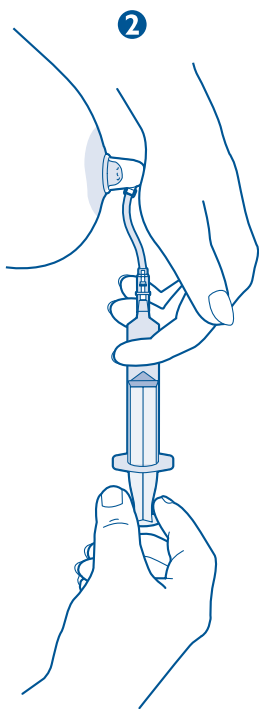

**Nipplette**<sup>TM</sup>

CE  
**PHILIPS**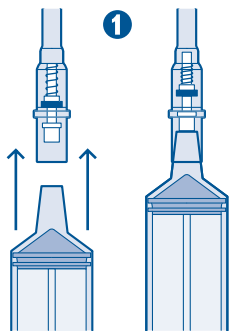

Inverted or flat nipples are the result of short lactiferous (milk bearing) ducts. The Nipplette™ is designed to lengthen these ducts by the application of gentle suction. It is painless and easy to use. Until recently, the only effective method of correcting inverted nipples was surgery, a costly solution that ruled out breastfeeding for good. The Nipplette quickly corrects inverted nipples and has proven to be a permanent solution for both cosmetic purposes and normal breastfeeding.

### When can you use the Nipplette?

- ◆ The Nipplette is suitable for correcting inverted nipples which have developed as a result of hereditary factors or during puberty.
- ◆ The Nipplette is suitable for patients having had unsuccessful surgery to obtain cosmetic correction.
- ◆ The Nipplette is NOT suitable for lactating breasts, however there is no harm in using the Nipplette for a few minutes to draw out the nipple and help your baby latch on.

**NB:** Use is not recommended during the last trimester of pregnancy.

**DO NOT** use on inverted nipples developing later in life without medical advice. If in doubt consult your doctor.

### Instructions for use (see diagrams)

- ① Push the syringe firmly into the open end of the valve.
- ② Hold the Nipplette over the inverted nipple with one hand and apply suction by pulling the syringe with the other. Pull as firmly as is comfortable. Do not pull too hard – a small smear of baby lotion at the base of the Nipplette will improve the seal and suction.
- ③ When the nipple has been pulled out, let go of the Nipplette, hold the valve and

carefully separate the syringe from the valve.

- ◆ You are now free to continue your daily activities wearing the Nipplette. It is easily concealed under loose clothing.
- ◆ To remove the Nipplette, push the syringe into the valve to release the vacuum.

### Important advice

- ◆ To obtain the best and swiftest results the Nipplette should ideally be worn as much as possible during the day and night and at the maximum suction comfortable for you. One Nipplette is required for each inverted nipple. It is possible to wear the device overnight provided you do not sleep on your front and it may be convenient to wear a loose fitting bra.
- ◆ Used regularly, the nipple should fill the mould within three weeks for most individuals. If the ducts are tight this may take between three to twelve weeks. Continue to use the Nipplette until the nipple does not retract. Use of the Nipplette can then be reduced. Any initial tendency to retract is easily overcome by intermittent usage.
- ◆ Excessive suction force applied to the nipple could cause bleeding to occur. Some individuals may notice a small amount of discharge from their nipples, this is entirely normal. If you experience more discharge or pain, decrease the amount of suction and consult a medical professional.
- ◆ If required the Nipplette can be cleaned in warm, soapy water. **DO NOT** sterilise.

### We're here to help...

For details of the full Philips AVENT range of breast and bottle feeding accessories, plus useful advice and information,  
**GB:** Call FREE on 0800 289 064  
**IRL:** Call FREE on 1800 509 021  
**AU:** 1300 364 474

[www.philips.com/AVENT](http://www.philips.com/AVENT)

**Alternatively write to address at the end of this leaflet.**

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Niplette™ Made in England. Hecho en Inglaterra.  
Manufactured by AVENT Ltd, Suffolk, CO10 7QS, England  
(AVENT Ltd is a member of the Philips Group)  
**[www.philips.com/AVENT](http://www.philips.com/AVENT)**  
under licence from Niplette Ltd.

Philips of Holland. All logos brands and product names are trademarks or registered trademarks of their respective holders and are hereby recognised as such.